


Quick Operation Guide

Dragon-CR435S B
RIC hearing aid




Please download and install "HA-Fit" APP first.


Hearing Aid

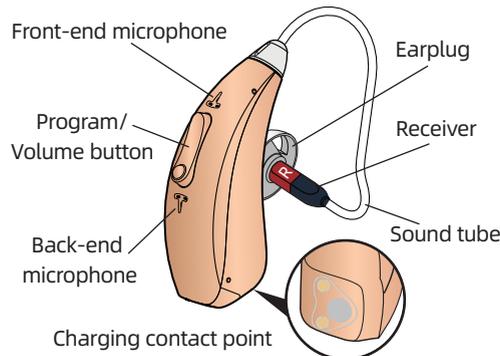

Accessories

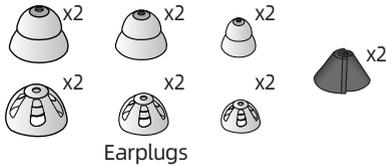


Charging Box

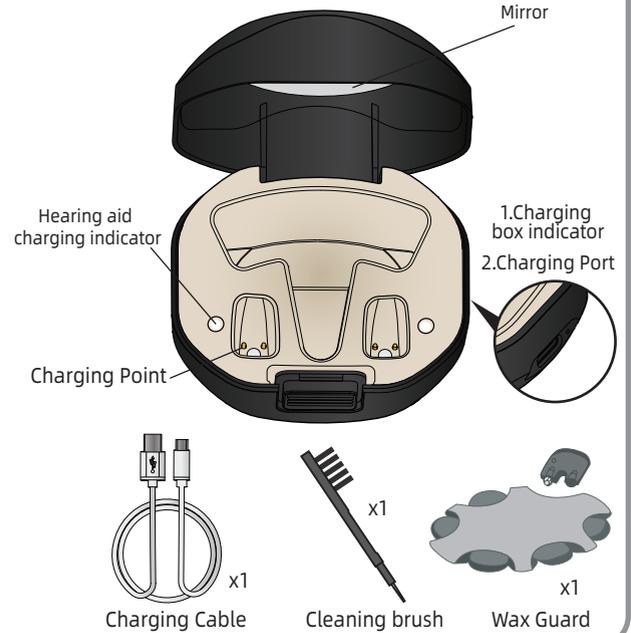


Prepare for Use

Charge the Device

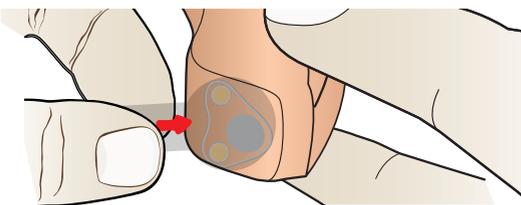

1 Remove the film

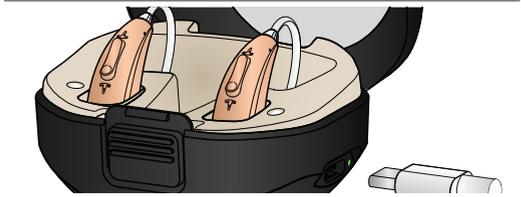


Hearing aid charging indicator


L	White light	R	L	Light off	R
	Charging			Full charged	

2 Put the devices into the charge case then connect they pe-C/USB cable to charging

Wearing Test

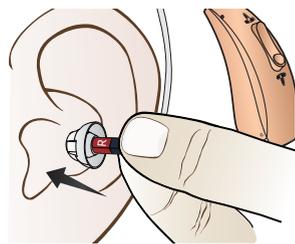

1 Gently insert the earplug into the ear canal.

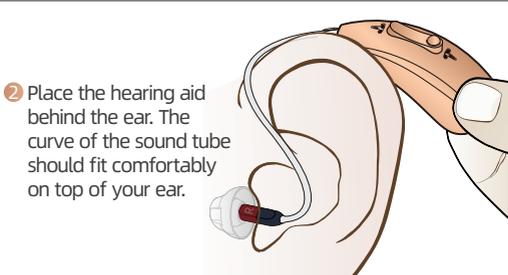


2 Place the hearing aid behind the ear. The curve of the sound tube should fit comfortably on top of your ear.

If the earplus not fitting, replace it.

Replace the earplugs

1 Choose a fitting dome

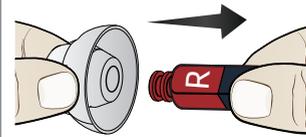


2 Insert the earplug straight and insert it gently



3 And then try to pull the earplug lightly to make sure that it will not come off


Use Guidance

Power on / off

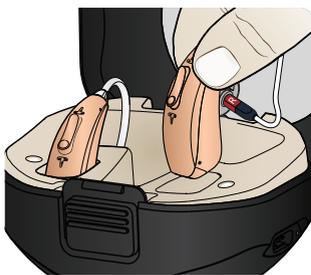

Automatic on

Turn on--device be takeout of the charge case

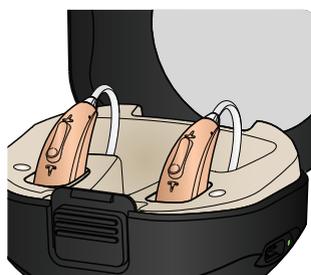


Automatic off

Turn off--device be putted into the charge case

Mode / Volumes

Press- the upper button to increase the volume

Press- the down button to increase the volume

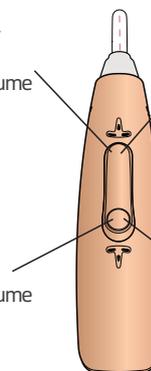


Long press mode toggle increment

Long press mode toggle decreasing

Press	3 beeps indicator the MAX or MIN volume level
Long Press	1 beep means mode 1(Indoor mode) 2 beeps mean mode 2(Noise cancelling mode) 3 beeps mean mode 3(Outdoor mode)

Low Battery Reminder

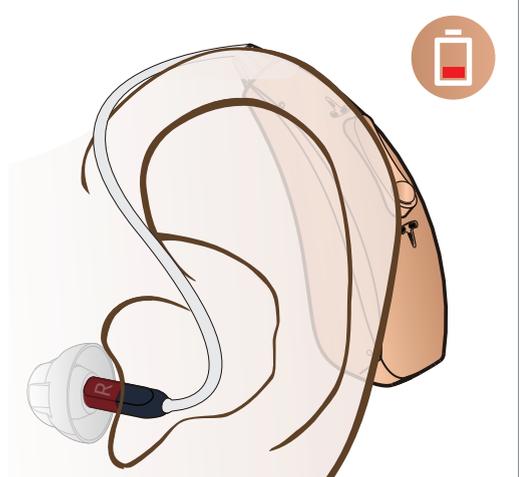


There are low battery warning signal beep sounds every one minute until the battery runs out. So please charge the hearing aids when you hear the beep.

[HA-Fit APP]

How to control APP

Please download and install "HA-Fit" APP first.

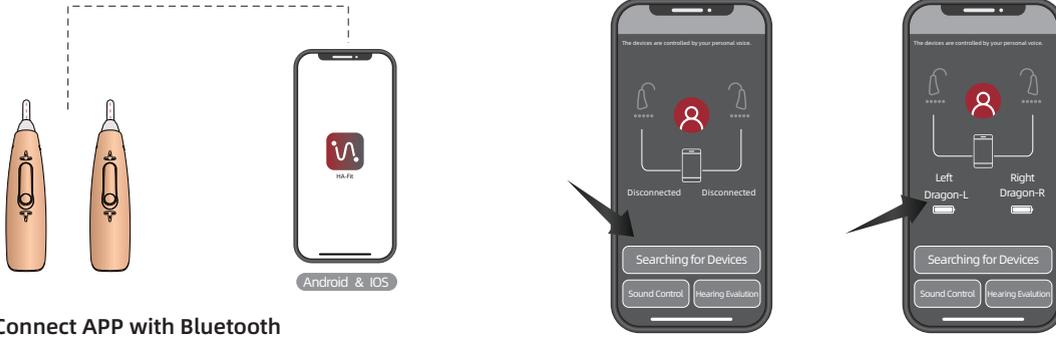


Connect APP with Bluetooth

- Step 1: Please turn on the hearing aid.
- Step 2: Be sure Bluetooth is enabled on your phone.
- Step 3: Open the "HA-Fit" APP.
- Step 4: Click the "Searching for Devices" button.

After "searching for devices", the HA-Fit APP will automatically connect to the hearing aid, and the successful connection status is shown in the figure. (Connected "Dragon..." which also can be customized as other name)

Hearing Evaluation

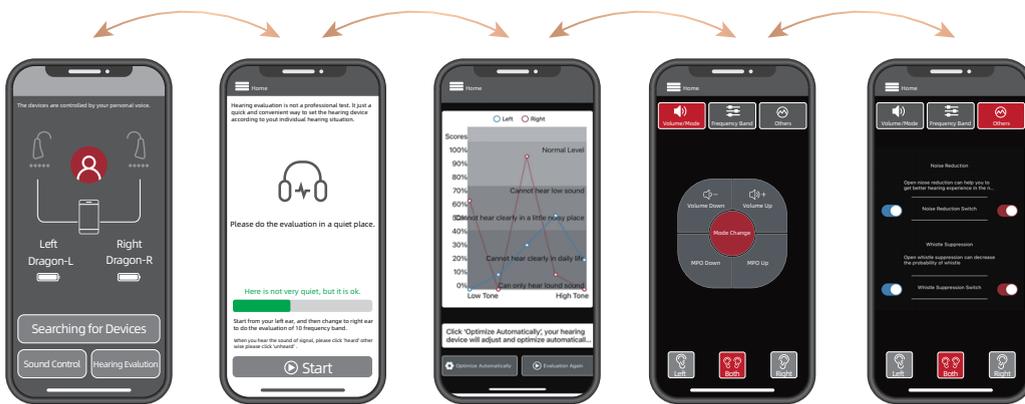

Get your hearing tested and optimize it accordingly

- 1 Please click "hearing evaluation" to star your hearing loss test in a quiet room according to APP system directives.
- 2 Follow the prompts to complete your hearing evaluation.
- 3 Click 'Optimize Automatically' to reset the device based on your finished hearing evaluation.

Adjust the volume, modes and MPO:

1. Press the button on app directly
 2. Adjust volume: Short press the button on device
 3. Adjust modes: press Mode Change on the button on device to change programs.
- P1: quiet, P2: conversation, P3: outdoor

Turn on Noise Reduction Function

Please turn on the function of Noise Reduction and Whistle suppression for hearing clearly


Please click "hearing evaluation" to star your hearing loss test in a quiet room according to APP system directives.

Daily maintenance

Care and maintenance


Keep the hearing aids clean at all time. After use, remove the earplug and clean the sound hole with the brush. Wipe off oil and moisture with tissue paper, etc., and keep the product dry.

1. Clean the sound hole with the brush.

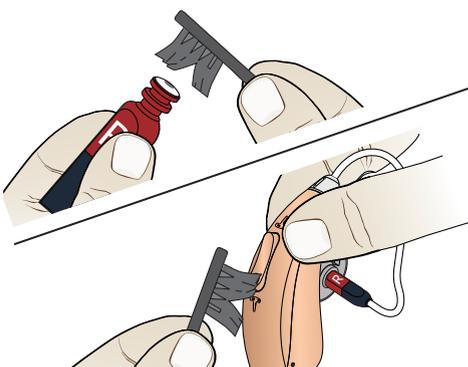

2. Clean the microphone with a cleaning brush.



3. Clean the charging case with a cleaning brush to charge the electric shock.

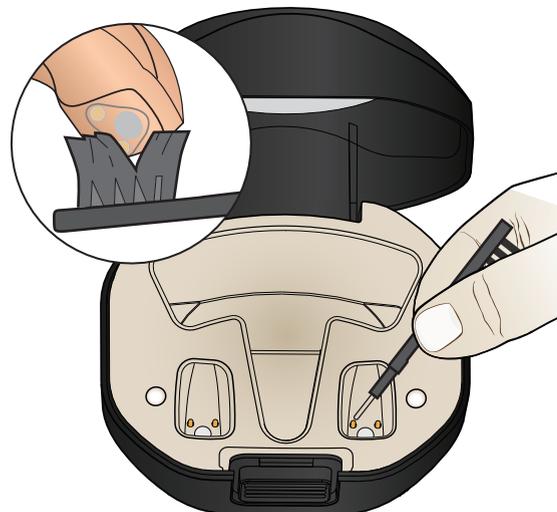


How to replace the earwax guard

The wax guard filter net is located in the sound hole of the hearing aid which can effectively prevent the cerumen from entering into the hearing aid to cause damage to the hearing aid. Replace the earwax filter every 3 months. Do not reuse the used filter.


Remove the earplug first. Refer to "How to replace the earplug".

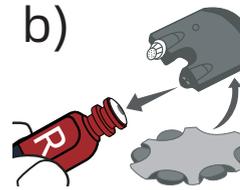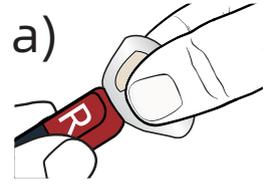


Align the removed end of the wax guard filter with the sound hole of the hearing aid and insert.

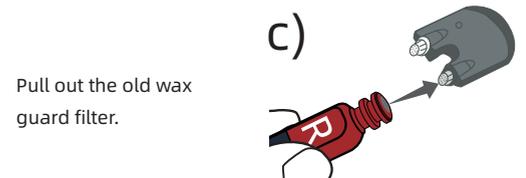

Pull out the old wax guard filter.

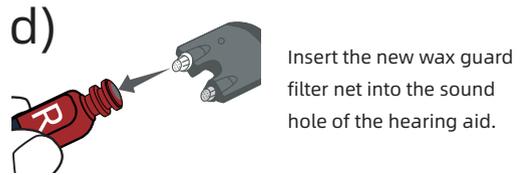

Insert the new wax guard filter net into the sound hole of the hearing aid.

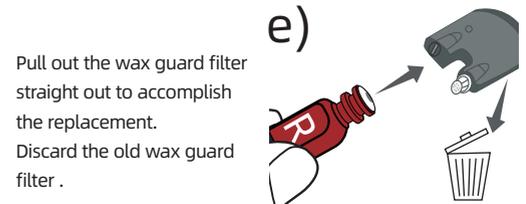

Pull out the wax guard filter straight out to accomplish the replacement. Discard the old wax guard filter.

Troubleshooting

Problem	Possible Solution
Sound is distorted	① Decrease volume. ② Charge the product. ③ Clean the dome.
The product emits whistling sounds	① Reinsert the Earplugs until it fits tightly. ② Decrease volume. ③ Clean the Earplugs.
The product does not work	① Change to different size of Earplugs. ② Check whether the microphone is damaged or not. ③ Charge the product.
Can't charge	① Clean the charging contacts. ② Make sure that the charging contacts are in good contact.

Consult your Hearing Care Professional or directly contact to us if the problem cannot be solved.

Notes and Tips

1. This product is not waterproof.
2. If the hearing aid is not used for a long time, please charge it every 1-2 months to avoid damage to the battery.
3. The adapter provided in the package is only for hearing aids, please do not use it to charge other devices
4. Store in a well-ventilated and dry place.
5. Do not use in or near an X ray or MRI machine.

All pictures are for reference only