



# USER GUIDE

Hearing innovations-Living innovations

King Series



## ***Thank you for purchasing our hearing aids!***

### Warning!


Hearing aid is a kind of device for hearing loss people, which are used to customization and fitting according to the individual hearing situation and listening habit. Please read this manual carefully firstly, and operate under the guidance of this manual. The hearing aids may be damaged under the improper operation.

# CONTENTS

Product structure.....	1
Attach your hearing instrument parts.....	2
Load the battery.....	3
Turn on and off.....	4
Volume Control.....	5
Adjust the program.....	6
Programming interface.....	7
Low Battery Warning.....	7
How to wear the hearing aids?.....	8
Change the battery.....	9
Battery Warning .....	10
Product maintenance.....	11

# Product structure

1. Programming button
2. Programming interface
3. Volume controller
4. Battery door
5. microphone
6. Earhook
7. Instrument

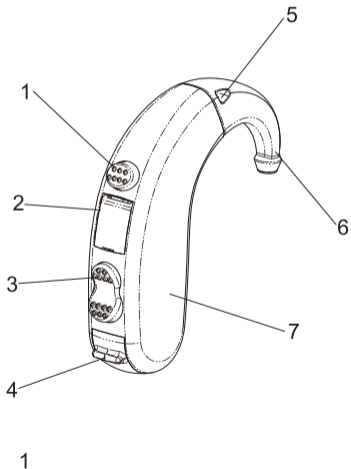

# Attach your hearing instrument parts

## Install Earplugs




Push the dome over the grooves on the tube.

## Cut the sound tube

The suitable length of the tube depends on individual ear size. We recommend cutting a small amount (1 mm or 1/16th of an inch) at a time so that you will not cut the length too short.

## Attach the sound tube

Hold the end of the sound tube in one hand and the hearing instrument in the other. Twist the tube onto the hearing aid.

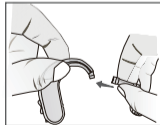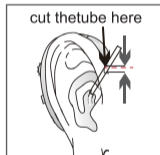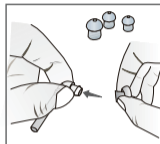

# Load the battery


The hearing instrument fits a size 13A Zinc-Air battery.

1. Open the battery compartment completely, using your fingernail to pull back on the small ridge located at the top of the door.

2. Insert the new one with the “+” (flat) side up. Make sure the battery has been placed correctly into the slot.

3. Close the battery door. Never use force to close the door as this may damage your hearing instrument.

**NOTICE:** When the battery door is completely closed, your hearing instrument is ready for use.

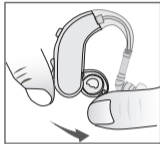


# Turn on and off

Your hearing instrument is equipped with an on/off switch integrated into the battery compartment.


## Turn on:

Turn your hearing instrument ON by closing the battery door.

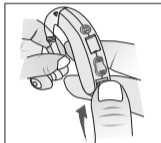

## Turn off:

Turn your instrument OFF by opening the battery door.

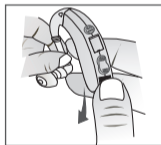

## NOTICE:

Turning your instrument off when you are not using it will increase the battery life. At night, turn off your instrument and open the battery door completely. The moisture will evaporate to increase the life span of hearing aids.




# Volume Control

This BTE models have a volume controller which allows you to set and control the volume manually. The rocker VC serves as the volume control. The volume control operates as described below:

**Press up for increase the volume.**

**Press down for decrease the volume.**

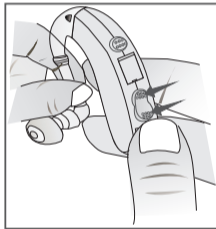


## NOTICE:

Please note that your instrument has been adjusted by your hearing care professional. Your instrument has volume remember function. The instrument reverts to the previous setting of volume when it power on again.

# Adjust the program

There is a button to adjust the 3 different programs.  
Each of them fit different environment.

- |                       |          |
|-----------------------|----------|
| 1. Quiet              | 1 beep " |
| 2. Noise environment  | 2 beep " |
| 3. Wireless coil mode | 3 beep " |

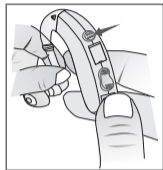

1.The program will move to the next program by each press.


The sequence is 1, 2, 3,1 ..

2.Your hearing instrument will alert to identify which program you are using. When you select program 1 you will hear one beep, program 2 will emit two beeps.

3.Please note that the programs are pre-set by your hearing healthcare professional or manufacturer.

# Programming interface

CS44 Programming interface

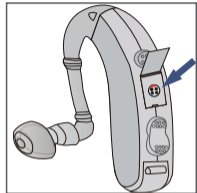


## Low Battery Warning

The hearing instrument has a soft low-battery-warning signal (beep-beep) that you will hear when the battery voltage/power gets too low. The signal indicates you should replace the battery. We recommend that you always carry a spare battery with you. The low battery warning signal will be repeated every 5-10 minutes until the battery runs out of power. The instrument will then automatically turn off.

## How to wear the hearing aids?

- Open the battery door to turn the instrument off. Hold the ear mould (or eartip) between your thumb and index finger to put it your ear canal. Now, slide the ear mould (or eartip) all the way into your ear with a gentle movement.
- Place the hearing instrument behind your ear. Move the ear mould(or eartip) up and down and press lightly to ensure it is positioned correctly in the ear. Opening and closing your mouth can ease insertion. You will feel when the ear mould(or eartip) is positioned correctly.
- After the hearing aid is correctly positiond, please turn on the hearing by closing the battery door.

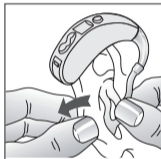

# Change the battery

Open the battery compartment completely. After removing the old battery, insert the new one with the “+” (flat) side facing the “+” on the battery door. Make sure the battery has been placed correctly into the slot. Never use force to close the door as this may damage your hearing instrument.



Always use a size 675A Zinc-Air battery. Please note that if the battery is weak, the hearing instrument performance will be reduced. Removing the battery when you are not wearing the instrument will help prevent corrosion of the battery contacts and prolong battery life.

# Battery Warning

Batteries contain dangerous substances and should be disposed carefully for your safety and the environment.

- Do NOT attempt to recharge batteries which are not specifically designated rechargeable as they may leak or explode.
- Do NOT attempt to dispose of batteries by burning them.
- Keep batteries away from small children and pets.
- Do NOT place batteries in your mouth.

# Product maintenance

•First, remove the tubing and ear mould (or eartip) from the hearing instrument. Keep left and right instrument separated.

•Remove earwax with the cleaning brush and a soft cloth. If necessary, use a mild solution of soft soap and water or a special cleaning tissue. Ask your



hearing professional for detailed instructions.

- Rinse the ear mould(or eartip) with water.

Note: Do not put water or other liquids on the hearing instrument.

- Dry the ear mould with a cloth.


- Remove water drops from the tubing and ear mould(or eartip). A special device is available for this from your hearing care practitioner.

- Ensure that the ear mould(or eartip) and tubing are completely dry before attaching them to the hearing instrument. Make sure that you connect the correct ear mould(or eartip) to their corresponding instruments.

## Precautions and warnings

- 1.Moisture proof, shockproof and high temperature resistant.

2. When not in use, please remove the hearing aid and open the battery box to turn off the power.

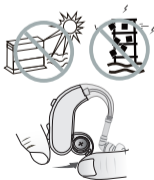

3. Do not wear the hearing aid when you are swimming, washing your hair or bathing. Please wear the hearing device when the ear canal is dry.



4. The outer surface of the hearing device should be kept clean and the earwax in the earplugs should be cleaned frequently.



5. Please adjust the keys slowly to avoid permanent damage.



6. This product should not be used at a high volume for a long time. In order to prevent damage to your hearing, when you feel uncomfortable, please stop using it.



7. Do not drop or hit the hearing device, which may cause the permanent damage.



8.If you hear a squealing sound,please check if the size of the earplug is suitable. Please select the appropriate size of earplug and plug it tightly, to avoid the air leakage.



9.Do not disassemble the hearing device by yourself.



10.Avoid electromagnetic radiation. In case of high frequency thermal therapy or operation,radiation therapy or strong magnetic field environment, please do not use the hearing devices.



11.The hearing device equip with batteries, the disposal must comply with local regulations.



12.Children are not allowed to play with the hearing device or the accessories. It may hurt themselves or others, or get the hearing device damaged.



# Learning to adapt your hearing aid:

## Step 1: Avoid stress on hearing

Start hearing training in a familiar environment and familiarize yourself with how to use hearing aids. During the first few weeks, do not wear hearing aids in noisy places, such as supermarkets or airports.

## Step 2: Slowly increase the time of wearing hearing aids

The initial period of wearing a hearing aid should not exceed one to three hours a day. At the same time, increase the time of wearing a hearing aid every day so that you can slowly adapt to the sound of the surrounding environment.

## Step 3: Learn to listen in a noisy environment

This world is a noisy place. You need to walk into a noisy environment, practice and train yourself to ignore those irritating sounds, then your task will be completed.

## Step 4: Discover the joy of listening

After three steps of practice, you can try to do everything you want, and experience all the sounds in the world again. If you find that the effect of hearing aids is still not ideal after training, you should slow down the training process and see your audiologist or doctor for the help.

Manufacturer:

Digital Soundlink GmbH  
Henri-Dunant-Strasse 102

81058 Erlangen, Germany

listen to life

---