



User Manual MINI-CN125

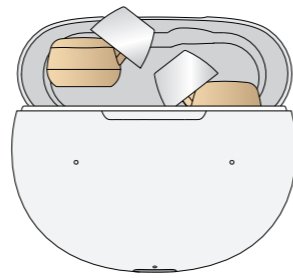

This guide contains important information, please read it carefully before using the product

Version: 052022

Parts List

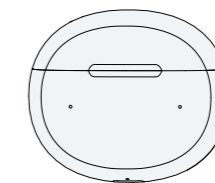


Charging case



screwdriver



Earplugs

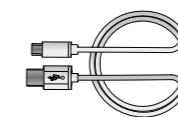

USB cable



Wax guard filter




cleaning brush



Getting started guide

1

Product description

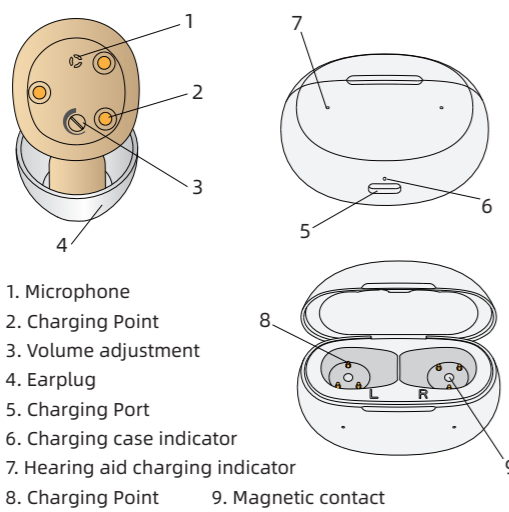


1. Microphone
2. Charging Point
3. Volume adjustment
4. Earplug
5. Charging Port
6. Charging case indicator
7. Hearing aid charging indicator
8. Charging Point
9. Magnetic contact

2

Instructions for use

Turn on

Take the hearing aids out of the charging case and it will turn on automatically after 5-10 seconds.

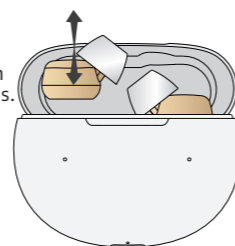


Turn off

Put the hearing aids into the charging case and it will turn off automatically

Volume adjustment

Turn clockwise to increase the volume;
Turn counterclockwise to decrease the volume;

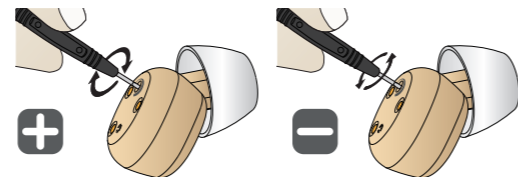

3


Instructions for use

Charge hearing aids

Put the hearing aid into the charging case, and the hearing aid will automatically start charging.

Charging: Orange light

Full charged: Lights off

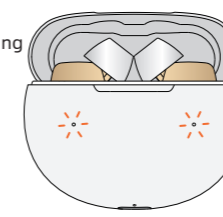


Charge the charging case

Connect the type-c charging cable to charge the charging case.

Charging: Flashing green light

Full charged: Green light

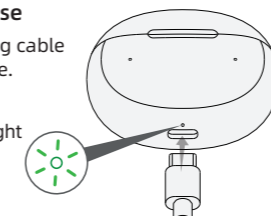


Hearing aids and charging case can be charged at the same time.

4


Earplug replacement guide

In order for users to have a better wearing experience, we presented 3 pairs of earplugs of different sizes. When your earplugs cause problems such as ear canal swelling, hearing aid whistling, etc., you can try to replace the earplugs.


1. Pinch the hearing aid body with one hand and other hand remove the unsuitable earplug.

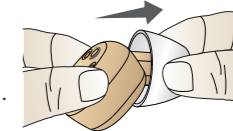

2. Choose the suitable one from the different sizes of earplugs.

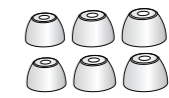


3. Align the appropriate earplug with the sound hole of the hearing aid and insert it, and make sure that they are tightly fixed.

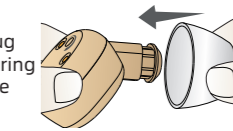


5

Wear instructions


1. To avoid howling when wearing, you can use a screwdriver to turn the volume knob to proper position that make you comfortable

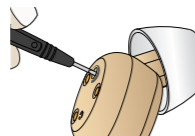

2. Pinch the body of the hearing aid and keep the earplug facing diagonally downward, aiming at the ear canal.

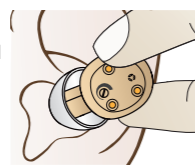

3. Put the hearing aid into the ear canal and twist the hearing aid up and down until the ear feels comfortable.

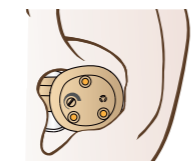

6


Daily cleaning and maintenance

Hearing aids are sophisticated electronic instruments. In order to prolong their service life, attention should be paid to keeping hearing aids clean.

1. clean the sound entrance with a cleaning brush.



2. Use the cleaning brush to clean the charging contacts.

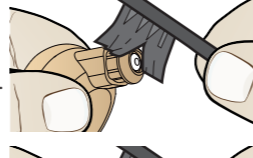

3. clean the outlet of the sound with a cleaning brush.




7


Wax guard Replacement Guide

The wax guard filter net is located in the sound hole of the hearing aid which can effectively prevent the cerumen from entering into the hearing aid to cause damage to the hearing aid. It needs to be replaced every once in a while.

1. Remove the earplugs, align the disassembly end of the wax guard filter with the sound hole of the hearing aid and insert it, and pull out the old wax guard filter.

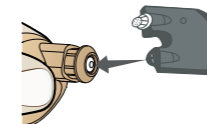

2. Insert the new wax guard filter net into the sound hole of the hearing aid.

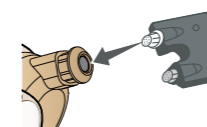

3. Discard the old wax guard filter.



8

Troubleshooting

Problem	Possible Solution
Sound is distorted	① Decrease volume. ② Charge the product. ③ Clean the dome.
Sound is weak	① Charge the product. ② Increase volume. ③ Check the dome whether blocked or not.
The product emits whistling sounds	① Reinsert the Earplugs until it fits securely. ② Decrease volume. ③ Clean the Earplugs.
The product does not work	① Change to different size of Earplugs. ② Charge the product.

9

Basic parameters

Max.OSPL90	≤115dB
HF Avg.OSPL90	105±4dB
Max gain	≤35dB
HFA Gain	23±4dB
Noise Level	≤32dB
Frequency Range	F1≤400Hz F2≥4500Hz
Battery Current	≤3%
Total Distort	NA

10

Basic parameters

	Hearing Aid	Charging case
Charging Time	≤2h	≤2h
Rated Working Time	≥25h	8 Times for Single Unit 4 Times for Two Units

11

Notes and Tips:

1. This product is not waterproof, therefore the user must take care to prevent water from entering the product.
2. Do not use hairspray or hair dryer when wearing hearing aids.
3. When not in use for a long time, please put the product in the charging case and charge it once every 1-2 months to avoid damage to the battery.

12